


Inami

ELECTRICAL TABLE

MANUAL

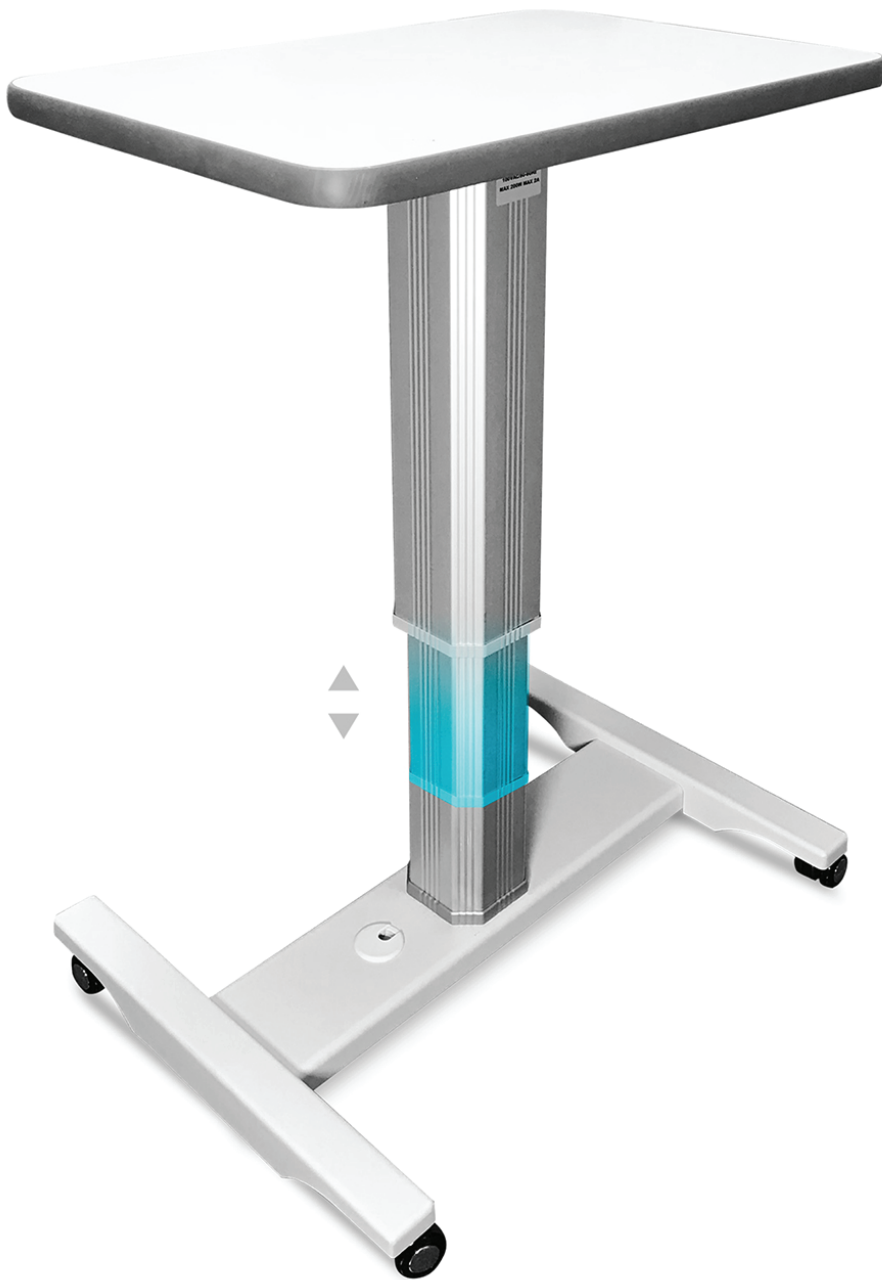

L-1425-PT

VIEWLIGHT
POWERED BY INNOVATION

Before use this instrument, be sure to read this manual

EN

CONTENTS

1. Safety	· · · · ·	1
2. Specification and component	· · · · ·	3
3. Part names	· · · · ·	4
4. Method of operation	· · · · ·	5
5. Maintenance	· · · · ·	5

1. Safety

For safer and more effective use of the apparatus, the reader is asked to observe the following points

General definitions of safety symbols are indicated below.





Caution



Denote the prohibitic



General mandatory a

 Caution	
	You must use the device by 200V AC. If you don't hold the voltage, you may get a electrical shock and take a fire.
	Don't attach the dust to power plug. If the dust attach, it may be fire.
	Connect the earth cable to the earth. If you don't connect, you may get a electrical shock and take a fire.



Caution

	<p>Don't touch the power plug with the wet hand. You have possibility of the electric shock.</p>
	<p>Don't attach the dust and the pin to plug. You have possibilities of the electric shock, the short circuit and the ignition.</p>
	<p>Don't remodel the device. Don't take apart and repair. You have possibilities of the ignition, the electric shock and the hurt. If you need the repair, please check the distributor.</p>
	<p>Don't use the device, when you find the damage of the cable and the plug. You have possibilities of the electric shock, the short circuit and the ignition.</p>
	<p>Don't injure and bend the cable, and don't put the heavy object. You have possibilities of the electric shock and the ignition.</p>



Caution

	<p>Don't take the cable when you pull the power plug. You have possibilities of the electric shock, the short circuit.</p>
	<p>Don't put the object more than 50kg on the table. You have possibilities of the falling and the breakdown.</p>
	<p>Please check the safety, when you change the height of table. You have possibility of the falling.</p>
	<p>Please put the object on the center of table. You have possibility of the falling.</p>

2. Specification and component


【Specification】

- 1) Table size
600mm(W) × 450mm(D)
- 2) Height
600mm ~ 850mm
- 3) Maximum acceptable weight
50 kg
- 4) Continuous running
150 sec
- 5) Rated voltage
AC220V
- 6) Consumed power
700W(outlet 400W)

【Component】

- 1) Main body (Attached Table, Up-Down switch and Caster)
- 2) Supply cable × 1
- 3) Conversion plug × 1
- 4) Fuse × 2
- 5) Manual × 1

3. Part names

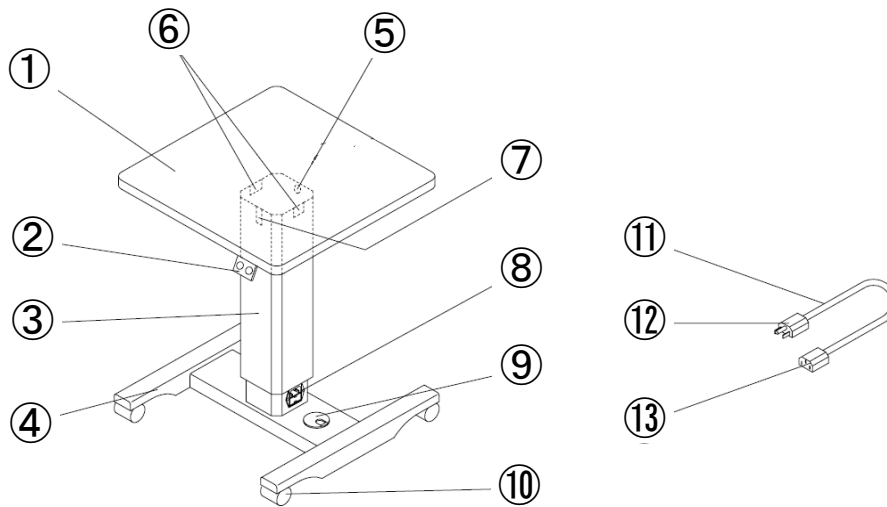

No.	Part name	No.	Part name
1	Table	8	Socket , Fuse box
2	Up - Down switch	9	Hole for the suply cable
3	Actuator	10	Caster
4	Base	11	Suply cable
5	Conector for the Up-Down switch	12	Power plug
6	Outlet × 2	13	Device plug
7	Pilot lamp		

4. Method of operation

① Please connect the cable between the device to the outlet, then the pilot lamp will turn on the light.

「▲」 Table up 「▼」 Table down

Note: Please push the switch to clip with the first toe and first finger.

② Please get the supply cable through the hole of base, and attach the cap.

③ Please push the lever of caster, when you want to fix the caster.

④ Please disconnect the power plug after using.

5. Maintenance

① Replacement of fuse

Please disconnect the supply cable and replace the fuse.

② Cleaning

Please wipe with the dry cloth. When it is heavy soil, please wipe with the damp cloth or using mild detergent.



Inami & Co., Ltd.

No.24-2, Hongo 3-chome,
Bunkyo-ku, Tokyo 113-0033, Japan
TEL.81-3-3814-5916 FAX.81-3-5684-2126