



L-0541DC-5 Digital Camera Unit 5M

1. Parts Composition

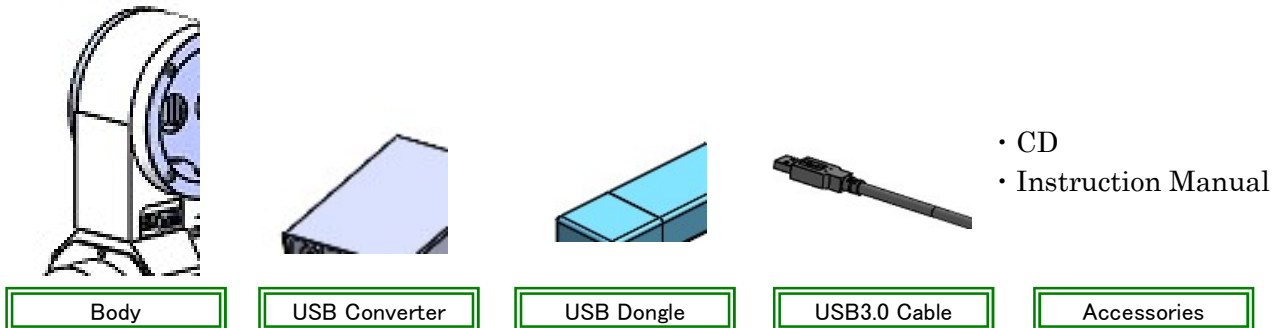


2. Installation Procedure

- Install the camera between the telescopic part and the variable magnification part.

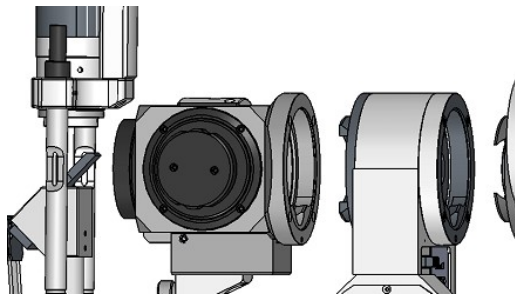


- Check the mounting orientation.
- Fix with the fixing screw.

- Connect the USB3.0 cable.

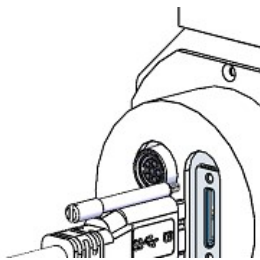

- Connect the USB cable to the USB port on the camera side.
- Connect the USB cable to the US B3.0 port on the PC side.

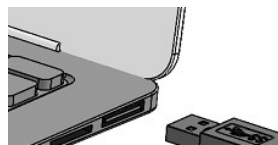


- Connect the USB dongle to your PC



- Connect the USB dongle to the USB port on the PC side.

3. How to use

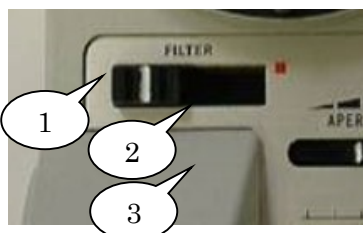

Functions of each part

- ① YELLOW FILTER: ←OFF ON→
- ② APERTURE: ←CLOSE OPEN→
- ③ FOCUS

※1. For how to use the software, refer to the PixelFiler 5.0 installation procedure and instruction manual.

※2. For how to use the USB converter, refer to the instruction manual of the slit lamp.

4. Specification

Model	L-0541DC-5
name	Digital Camera Unit 5M
OS	Windows10 64bit
CPU	8th Generation Intel Core Processors and Later Quad Core and above Processors
memory	8GB or more
Required free space	5GB or more
Number of pixels	5 MP
Image sensor size (inch)	1/2.8
shutter	rolling
Video output type	USB3.0 (USB3 Vision)
Product outline	90mm x 60mm x 150mm
Body weight (camera)	1kg
Power-supply voltage	USB 3.0 interface
Compatible connection USB	USB3_Cable_3M_S (3m)
Compatible image software	PixelFiler5.0



Inami & Co.,Ltd.

No.24-2, Hongo 3-chome,
Bunkyo-ku, Tokyo 113-0033, Japan
TEL.81-3-3814-5916 FAX.81-3-5684-2126